



SchureFoot TRK Ball Track System

Item #800-0141

User's Manual

**Schuerch Corporation
50 Messina Dr.
Braintree, MA 02184**

**(888) 724-8763
(781) 380-0905 Fax
www.schuremed.com**




SchureFoot TKR Track System User Manual

- This product is latex free and no latex is used during its production
- For product warranty issues contact Schuerch Corporation Customer Service at:

Schuerch Corporation
50 Messina Dr.
Braintree, MA 02184
(888)724-8763
(781) 380-0905 Fax
www.schuremed.com


It is recommended that with this patient positioning device you should become familiar with its feature operations. You should always practice its use on a nurse, physician or appropriate volunteer prior to using it clinically

SchureFoot TKR Track System Insertion and Operation

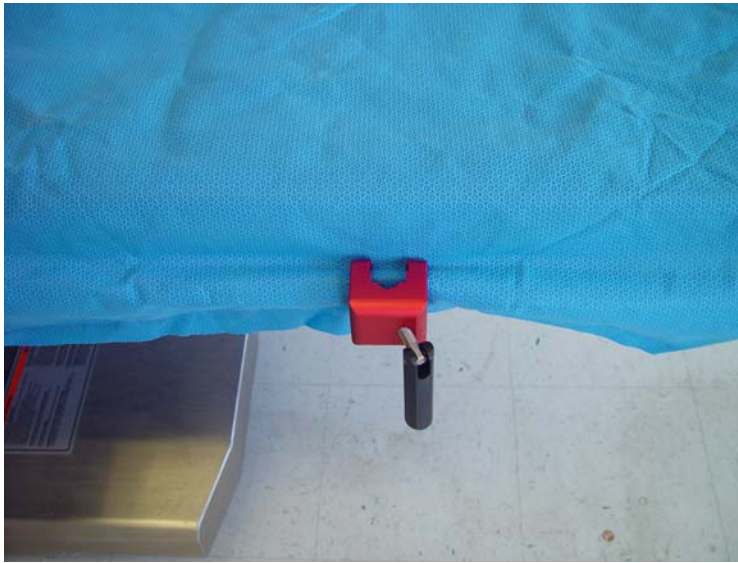


1. Drape the bed.


2. Attach the universal rail clamp (800-0085) to the rail.

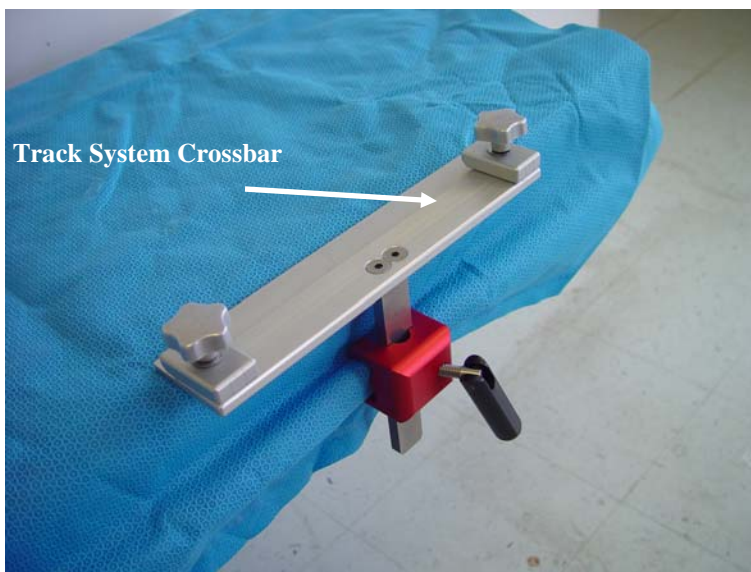


3. Slide Track System Crossbar onto the universal rail clamp, approximately 2" from patients heel. Securely tighten the Track System Crossbar by turning the handle on the clamp.

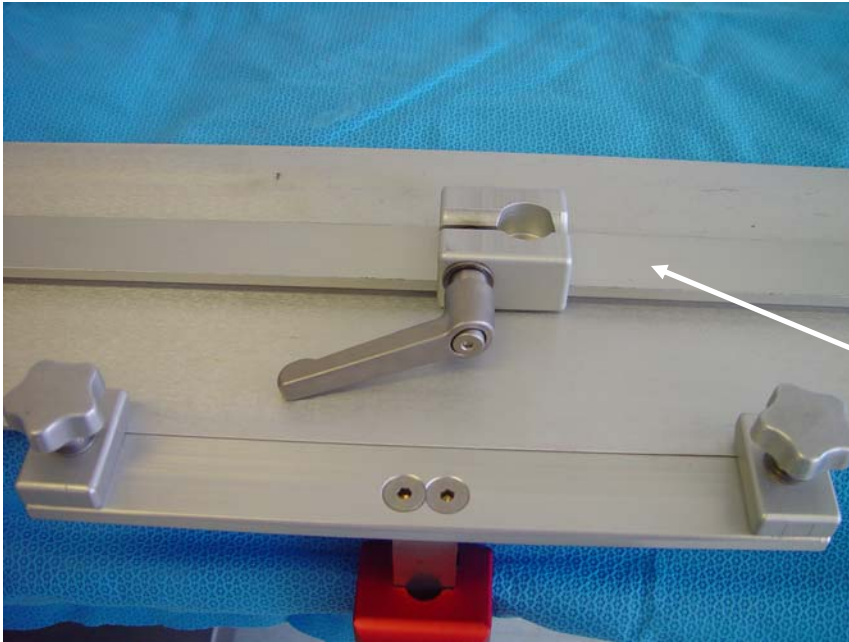

4. Slide the adjustable block on to the Track Sys-

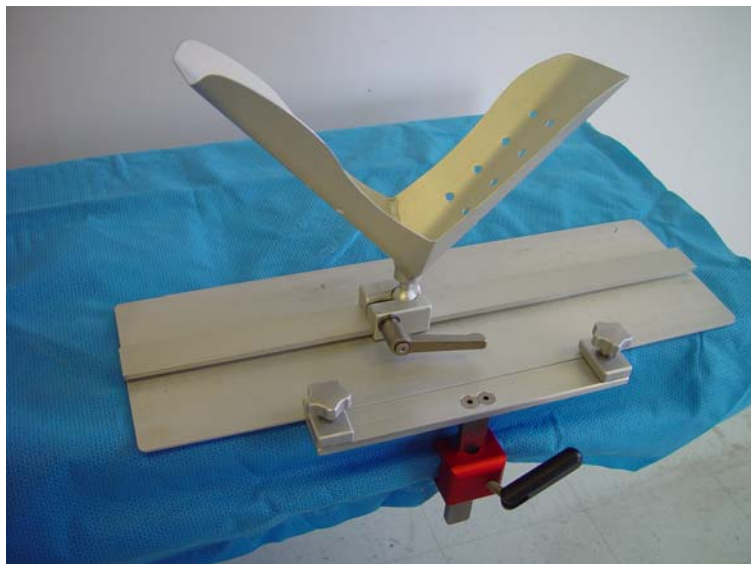

4. Place the boot into the Track Systems Adjustable Block

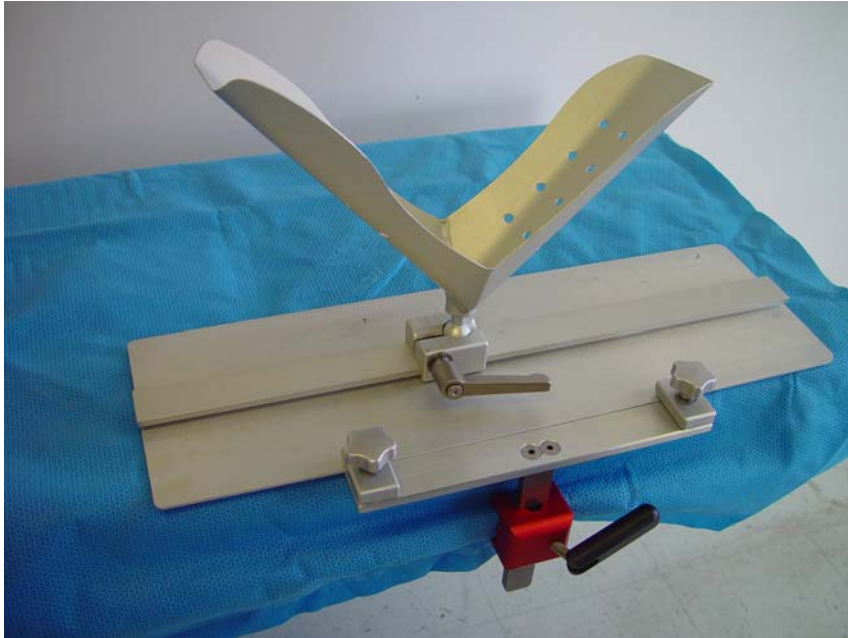

5. Line the boot with our Sterile TKR boot Liner (800-0146) or other suitable item for padding and engage the boot on the Track System. The Adjustable Boot Ball can be placed in the following positions: extension, flexion, tilt, and rotation and can be locked anywhere on the Track.