

Installation and User Instructions

Step #1 Place Pivot Bracket in desired position
Install with the retaining tabs upward
Attach with supplied hardware.



Step #2 Insert Hanger Rod thru the Nylon Bushings and slide the Plastic Tip over the end of the Hanger Rod



Step #3 Lift Hanger Rod, rotate against the wall and then lower into stowed position when not in use.



The bracket is made from 20 gauge steel and then it is powder-coated white. The rod is a stainless-steel 1/4" rod. The recommended maximum capacity for the assembly is 20 lbs. when the bracket is mounted securely to a stud or other solid structure. If you are mounting only into drywall with dry wall anchors, we recommend keeping the load less than 10-15 lbs because the quality of drywall/sheetrock can vary greatly.