

SW MED-SOURCE

PO Box 93115, Southlake, TX
 Toll Free: 1 888 424-1740
 Fax#: 817 424-1745

Stainless Steel Finger Trap

Finger Traction Device



Rx Only



REF: K3278, K3280, K3285, K3290



How to Use: Place patient's fingers in finger traps. If using stainless steel traps, fingers may be wrapped in sterile dressing or 3M Co-Ban to prevent unnecessary skin trauma. Once fingers are secure in traps, place the loop of the finger trap in the appropriate clip on ISI's Digi Grip or other traction device. The traction device may be suspended from an IV pole or traction system.

You may also use ISI's Digi Grip, Traction Strap and weight set for additional patient stability.

DG-3000	Digi Grip
DG-7550	Digi Grip System
WS1820	Traction Strap
S Hook	Stainless Steel S Hook
0350	½ pound (.023 Kilogram) weight
0352	1 ½ pound (.068 Kilogram) weight
0354	2 ½ pound (1.13 Kilogram) weight
0357	5 pound (2.27 Kilogram) weight

Sterilization Recommendations: Clean with any standard disinfectant prior to sterilization.



Sterilize the finger trap before each use

1. Steam Sterilization General Instructions

Follow standard hospital sterilization practices.

- A. Place Clean Finger Traps in an instrument tray.
- B. Follow the instructions below for recommended sterilization processes.

2. Pre-Vacuum Steam Sterilization

- A. Place Finger Traps in a CSR wrap prior to sterilization.
- B. Set temperature to 132 degrees Celsius.
- C. Set exposure time for 4 minutes.


3. Gravity Steam Sterilization

- A. Place Finger Traps (unwrapped) into instrument tray.
- B. Set temperature to 132 degrees Celsius.
- C. Set exposure time for 4 minutes.

After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect Finger Trap before and after each use.



If the Finger Trap is split or frayed, **DO NOT USE**. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

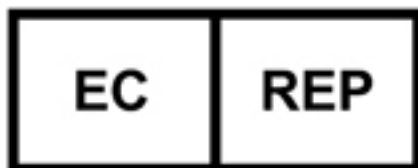

EMERGO EUROPE
 Molenstraat 15
 2513 BH, The Hague
 The Netherlands
 Tel: +31 (0)70 345 8570
 Fax: +31 (0)70 346 7299

Explanation of Symbols

Symbol	Explanation
	Manufacturer
	European Authorized Representative
	Non-Sterile
REF	Catalogue Number
	Lot Number
	Caution, Consult Accompanying Documents