

Nylon Finger Trap

Finger Traction Device



Rx Only



REF: K-100-XS, K- 150-XS, K-200-S, K-250-S, K-300-M, K-350-M, K-400-L
K-450-L, K-500-XL, K-550-XL, K-650, K-750



How to Use: Place patient's fingers in finger traps. Once fingers are secure in traps, place the loop of the finger trap in the appropriate clip on ISI's Digi Grip or other traction device. The traction device may be suspended from an IV pole or traction system.

You may also use ISI's Digi Grip, Traction Strap and weight set for additional patient stability.

DG-3000	Digi Grip
DG-7550	Digi Grip System
WS1820	Traction Strap
S Hook	Stainless Steel S Hook
0350	½ pound (.023 Kilogram) weight
0352	1 ½ pound (.068 Kilogram) weight
0354	2 ½ pound (1.13 Kilogram) weight
0357	5 pound (2.27 Kilogram) weight

Sterilization instructions: Clean with any standard disinfectant prior to sterilization.



Sterilize the finger trap before each use.

1. Steam Sterilization General Instructions:

Follow standard hospital sterilization practices

- A. Place clean instruments in an instrument tray
- B. If instruments are to be wrapped, two double thicknesses of #140 thread count wrappers should be utilized. Exposure times are the same for wrapped or unwrapped instruments
- C. Follow the instructions for the gravity air displacement steam sterilization listed below